REDO-CAST – Wofür?

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Tel: +49 2262 717 2841

www.allredo.eu

APPLICATION SUGGESTION FOR REDO-CAST

What is required?

2 x REDO-FLEX synthetic undercoat padding 4 m x 10 cm

2 x REDO-CAST adhesive bandage 7.5 or 10 cm (in another colour than the REDO-CAST Tape, e.g., Red)

2 x REDO-CAST Tape 10 cm x 3.6 m

2 x watertight disposable gloves (To wear before opening the REDO-CAST Tape)

1. Put on REDO-FLEX undercast Padding 4m x 10 cm:

Let us start first with putting on the undercast padding. This happens in the same manner as putting on an adhesive bandage. Nevertheless, in this case both sides of the claw are wrapped carefully and more times. Normally, 2 of the Redo-Flex undercast padding 4 m x 10 cm are sufficient in order to put a bandage on, which goes over the after claws. Important: Use synthetic Undercast padding to avoid suction of moisture. Take care, that you create enough padding, so that you will have a good distance between the claw and the cast tape bandage later. (This is important to remove it later with the grinder or oscillatory saw)

2. put on adhesive REDO-FLEX claw bandage:

In the next step REDO-FLEX bandages are wrapped around the whole REDO-FLEX Undercast Padding. These hold the cotton in position and defines the form to the REDO-CAST Tape, which you put on later.

Expert's tip: Take a coloured claw bandage REDO-FLEX in connection with a white REDO-CAST Tape. This will give you good guidance later when you cut open the REDO CAST Tape as you can see where the REDO-CAST finishes. It is also recommended not to wrap each single claw, but to bypass the gap between the two claws with the bandage. This will also help to remove the REDO CAST Bandage properly.

1. REDO-CAST put on Casting Tape:

The REDO-CAST Casting Tape is packed hermitical and waterproof. The package can be opened easily when you use the provided notch. Take the Casting Tape REDO-CAST and

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hold this for a short time under water and exert with your hands light pressure on the REDO-Cast Casting Tape while underwater to allow the water to go into every layer.

TIP: The warmer the water, the faster the REDO-CAST reacts

Put on the REDO-CAST in the same way as you did with the adhesive Redo-Flex. Here again it is recommended to bypass the gap between the two claws. Every place should be covered several times above each other with the REDO-CAST Tape to create a solid sole. At least 5-6 layers are required to get a stable Cast capable of carrying the weight of a cow.

Normally it is enough if the REDO-CAST is taped up to the after claws. If enough layers are put under the claw, the REDO-CAST tape holds 3-4 weeks until it is "run down".

TIP: To avoid cut injuries and pressure make cuts on both sides of the tape and bent the top 1-2 cm to the outside so that you create a little collar.

Info: About 5 Minutes after watering the Redo Cast Bandage, reaction will start and the bandage will become warm and starts to harden. After further 5-10 Minutes the Cast Bandage will already be resilient.

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